RADIOMETER 

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# DETERMINE™ HIV-1/2 TEST

Rapid HIV screening, tested by billions, proven by practice.

**EASY-PEEL DESICOVER  
PROTECTS TEST**

**CLEAR DEPENDABLE  
RESULTS IN JUST 15 MINS**

**SIMPLE PROCEDURE,  
EASY TO USE**

## QUALITY COUNTS

We know that quality counts and we know that quality isn't easy. Manufactured in Japan, the Alere Determine™ HIV-1/2 test is produced under the highest quality standards. For our manufacturing team, quality is a way of life and a commitment to provide the products you deserve, every time.

## COMPACT AND FLEXIBLE FOR STORAGE AND SHIPPING

Designed with a wide temperature range, providing you with flexible storage options to suit your environment.

Clever packaging design means easier storage and shipping. Small and lightweight, the Alere Determine™ HIV-1/2 test allows you to maximize your valuable storage space. The removal of unnecessary packaging materials greatly reduces shipping costs while also producing less waste, for a better impact on the environment.

100 TESTS = 106g

EASY TO STACK MULTIPLE KITS FOR COMPACT STORAGE

Accurate answers to difficult questions

# DETERMINE SYPHILIS TP TEST<sup>TM</sup>

PROVIDING YOU WITH THE TOOLS TO RAPIDLY  
DIAGNOSE SYPHILIS AT THE POINT-OF-CARE

## RAPID

Provides accurate and reliable results in just 15 minutes, ensuring testing and treatment of patients in the same visit.

## CONVENIENT

The test is easy to transport and store, with no refrigeration required and storage conditions of 2-30°C. No power or water source is needed to run the test.

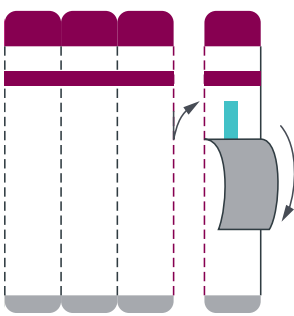
## FLEXIBLE

The test can use whole blood, serum or plasma.  
Collection can be by fingerstick or venipuncture.

## PROCEDURE


1

Prepare Test  
Tear one strip from the right and remove cover.

2

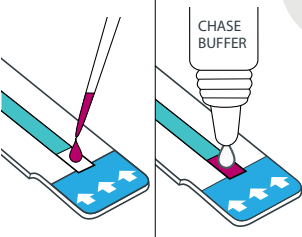Add Sample  
Add 50 µL sample to pad.  
If using whole blood, add one drop of chase buffer 1 minute after applying whole blood to pad.

3

Read Results  
Wait 15 minutes and read the results.

RESULT KEY

LINE	POSITIVE	NEGATIVE	INVALID
CONTROL			
PATIENT			

Refer to the User Guide and Product Insert for complete instructions.

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